

Exa® PACS RIS

Feature Summary

Teaching Studies

©2023 Konica Minolta Healthcare Americas, Inc.

Setup for teaching studies

Add user rights

1. **SETUP > OFFICE > User Management > USER ROLE >** Teaching Study

Configure an AE Title

Configure an AE title for teaching studies. Below is an example of an AE title that cycles back into the current system.

1. **SETUP > OFFICE > DICOM > AE TITLE > ADD.**
2. Verify ports are open. Common ports: 104 / 4334.

SAVE PING DICOM ECHO BACK

<p>AE Type *</p> <div style="border: 1px solid #ccc; padding: 5px;"> <input checked="" type="checkbox"/> SEND <input type="checkbox"/> PRINT <input checked="" type="checkbox"/> RECEIVE <input type="checkbox"/> QUERY/RETRIEVE </div>	<p>AE Flag</p> <div style="border: 1px solid #ccc; padding: 5px;">DICOM</div>
<p>AE Title (Remote) *</p> <div style="border: 1px solid #ccc; padding: 5px;">EXA_SCP</div> <input type="checkbox"/> Inactive <input type="checkbox"/> Ignore Scan Docs (Send)	<p>Institution</p> <div style="border: 1px solid #ccc; height: 20px;"></div>
<p>My AE Title</p> <div style="border: 1px solid #ccc; padding: 5px;">EXA_DEMO</div>	<p>File Store *</p> <div style="border: 1px solid #ccc; padding: 5px;">Exa Server</div>
<p>Host Name *</p> <div style="border: 1px solid #ccc; padding: 5px;">127.0.0.1</div>	<p>Port *</p> <div style="border: 1px solid #ccc; padding: 5px;">104</div> <input type="checkbox"/> Require SSL
<p>Description *</p> <div style="border: 1px solid #ccc; padding: 5px;">Send to Self / Teaching</div> <input type="checkbox"/> Disable QC2LIVE	<p>Transfer Syntax (Send)</p> <div style="border: 1px solid #ccc; padding: 5px;">JPEG 2000 Image Compression</div>
<p>Retries</p> <div style="border: 1px solid #ccc; padding: 5px;">0</div>	<p>Transfer Syntax (Receive)</p> <div style="border: 1px solid #ccc; padding: 5px;">JPEG 2000 Image Compression</div>
<p>Max Concurrent</p> <div style="border: 1px solid #ccc; padding: 5px;">1</div>	<p>Vehicle</p> <div style="border: 1px solid #ccc; padding: 5px;"> <input type="checkbox"/> Van 1 </div>
<p>Receive Facility *</p> <div style="border: 1px solid #ccc; padding: 5px;">DEMOAPPS x</div>	<p>SDE Modality</p> <input type="checkbox"/>
<p>Send Facilities</p> <div style="border: 1px solid #ccc; padding: 5px; background-color: #007bff; color: white;">ALL SELECTED (6)</div>	<p>Include Approved Report</p> <input type="checkbox"/>
<p>Send Annotations as</p> <div style="border: 1px solid #ccc; padding: 5px;">Burned in</div>	
<p>Send Unique Image UID</p> <input checked="" type="checkbox"/>	
<p>From Viewer</p>	
<p>Ignore Inbound Stat</p> <input type="checkbox"/> ?	
<p>Value</p>	
<p>Issuer Type *</p> <input checked="" type="radio"/> None <input type="radio"/> Static <input type="radio"/> Dynamic	

Configure a receiver rule

1. **SETUP > DICOM > RECEIVER RULES > ADD.**
2. Enter required settings. Select the AE title you created previously.

Rule Information and Filter

Rule Name * Inactive

AE Title

Facility

Modality

Institution

Status

Interval (Sec.) *

Trigger Routing Rules

Issuer Of Alternate Account Number


Receiver Rule

Applied Rules

- Reconciliation Mode: PACS Only Site
- Emit Live Update
- Precache Study
- Precache Priors
- No. of Priors: 3
- Select Study description from DICOM
- Patient
 - Account No./DICOM Patient ID
 - Last Name
 - Date of Birth
 - Move Anyway (Even in Exception)
 - Move to Live on No-Match
- Study
 - Study UID
 - Move to Live on No-Match

Create a teaching study

1. On the worklist, select the study you want to create as a teaching study.
2. On the study shortcut menu, hover over **Images**, and then select **Create Teaching Study**.

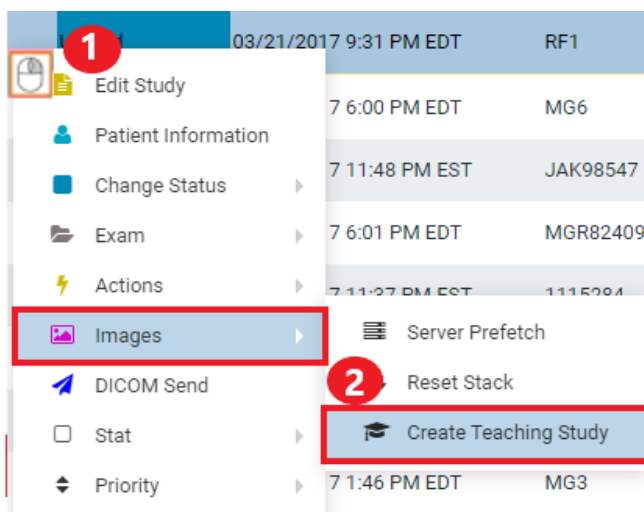

3. In the **Teaching Study** dialog, in the **AE Title** list, select the AE title.
Teaching studies can be sent to outside systems or back into Exa/Symmetry.
4. In the **New Value** column, type or select anonymous values for the new study.
5. Select **CREATE TEACHING STUDY**.
The Window automatically closes.

Teaching Study

3
AE Title
EXA_SCP (127.0.0.1 : 104)

5 CREATE TEACHING STUDY
CANCEL

Field	Current Value	New Value 4
Accession No.	91771	<input style="width: 95%;" type="text" value="uevz3Dn9ncuu80tP"/>
Patient ID	993543ca441bef2395ed92533ad51f96	<input style="width: 95%;" type="text" value="xUXHTgQgJg7pwn1m"/>
Patient Name	Patient3, Mammo	<input style="width: 95%;" type="text" value="Teaching_MG_204"/>
Study Description	3D Screening Mammogram	<input style="width: 95%;" type="text" value="3D Screening Mammogram"/>
Patient Gender	F	<input style="width: 95%;" type="text" value="Other"/>
Referring Phys. Name		[No Value]
Institution Name	VIZTEK	[No Value]
Institution Address		[No Value]
Station Name	ODCDIM1	[No Value]
Inst. Dept. Name		[No Value]
Operator Name		[No Value]
Device Serial No.		[No Value]

A message appears on the worklist stating that the teaching study will be created in 30 minutes. Most studies do not take that long to process.

Teaching Study will be created in 30 min