

Exa® PACS/RIS


Feature Summary

Opening Trauma CAD


©2025 Konica Minolta Healthcare Americas, Inc.

Open Trauma CAD from the Exa PACS/RIS worklist

1. In Exa, add an external apps button for Trauma CAD.


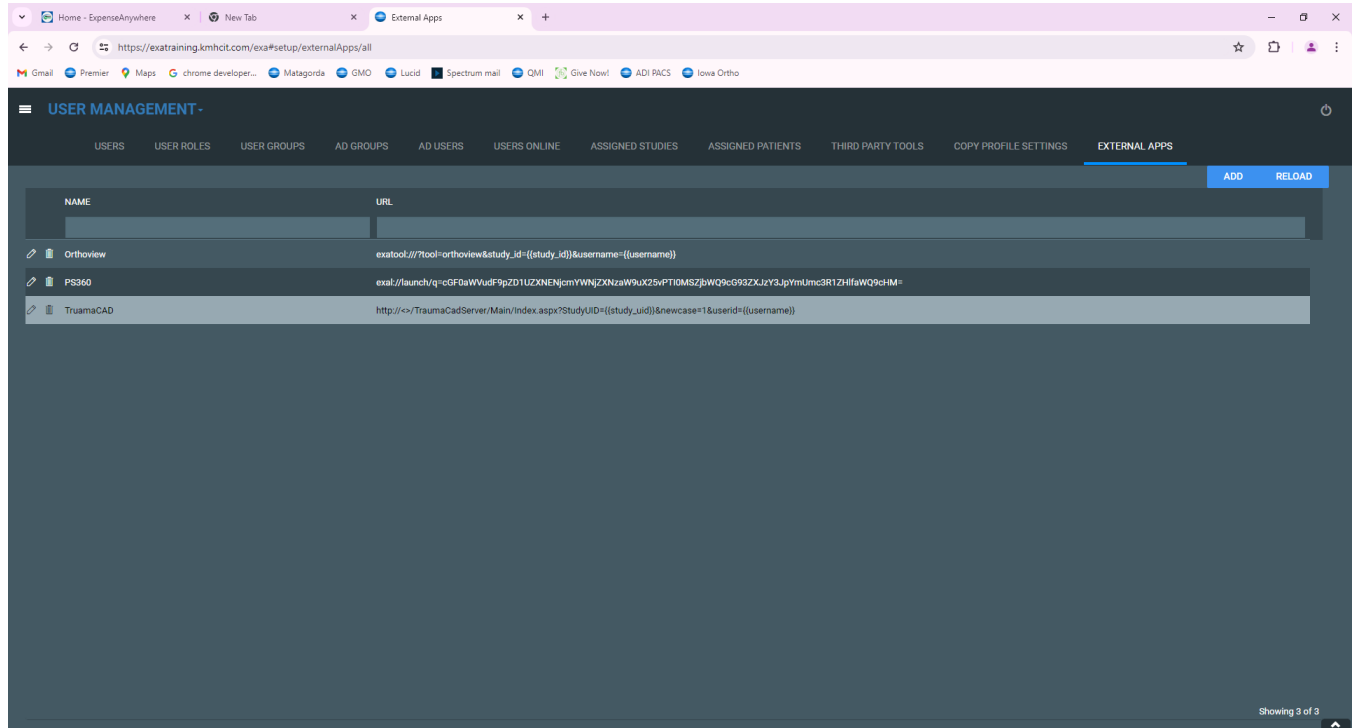


2. From the worklist, go to the study you need to template, select the external app button, and select **TraumaCAD**.

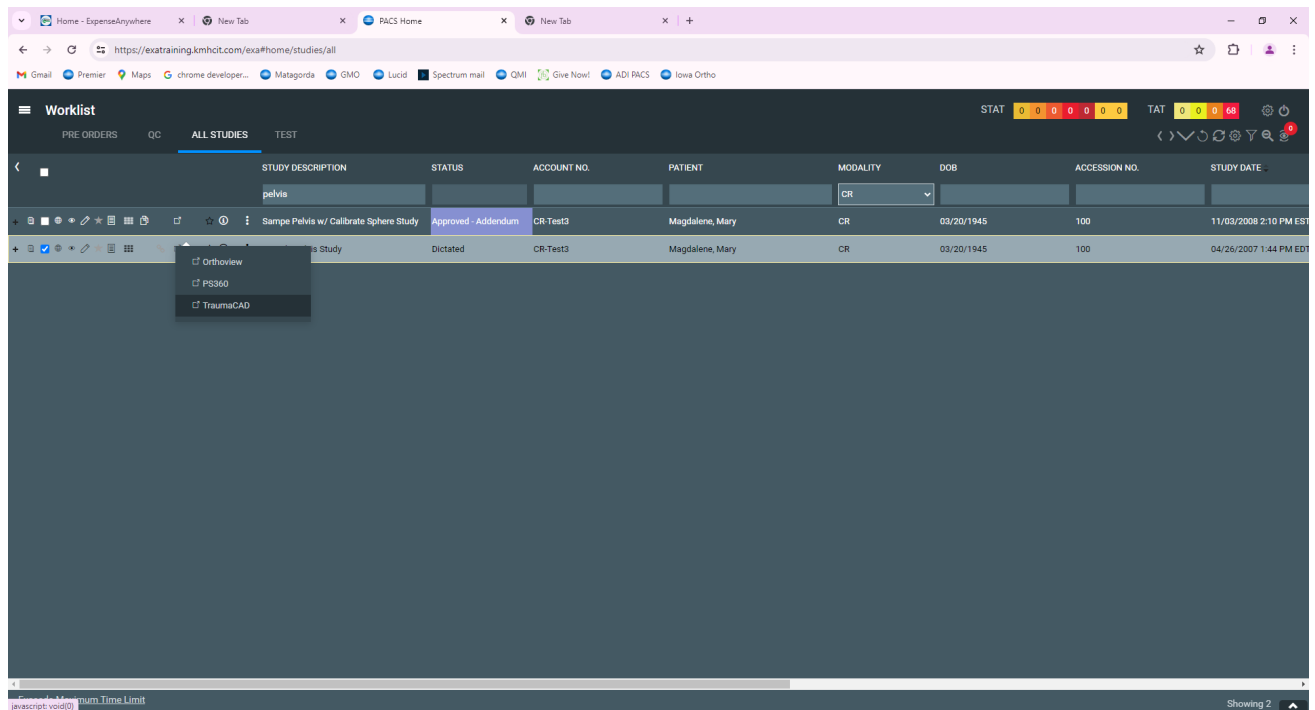

Open Trauma CAD from the viewer

1.) Go to **Viewer Settings > External Tools**, and add an entry for TraumaCAD.

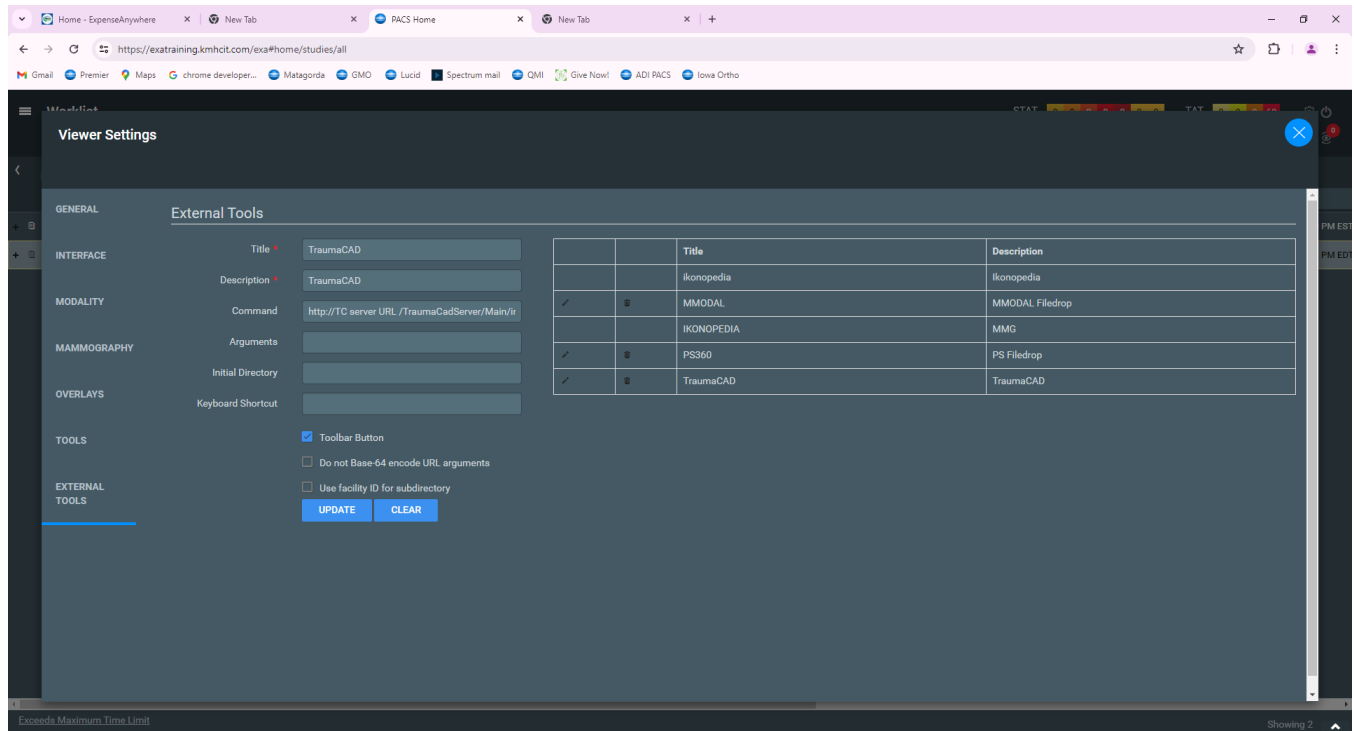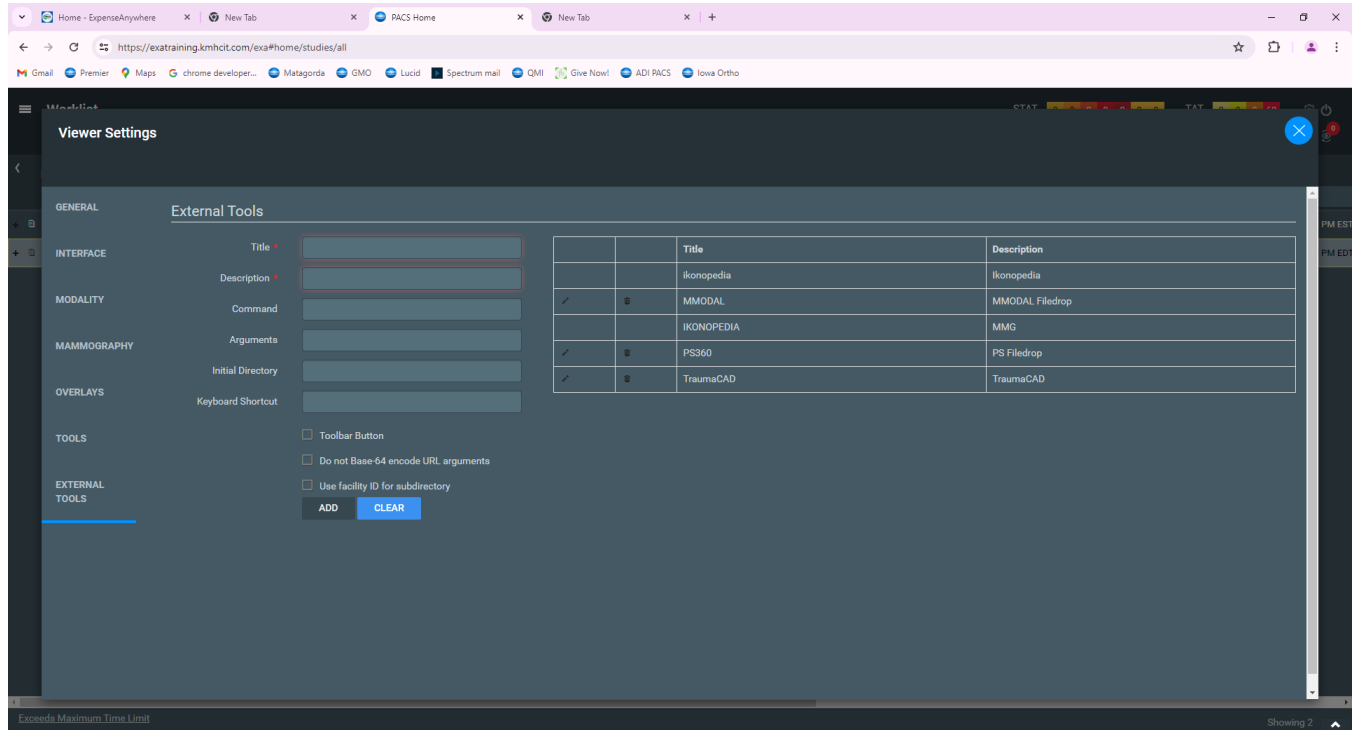

Command variable = http://TC server URL

/TraumaCadServer/Main/index.aspx?autostart=1&kill=1&studyuid={{study_uid}}